# Xailin Fresh

**Carmellose 0.5% w/v  
Eye Drops  
Single Dose Units**

PRESERVATIVE FREE

## What is Xailin Fresh?

Xailin Fresh is an eye drop containing carmellose 0.5% w/v in 30 x 0.4 ml or 5 x 0.4 ml (sample) preservative free single dose units that is sterile until opened.

## Composition:

Xailin Fresh contains Carmellose Sodium 0.5% w/v, Sodium Chloride, Calcium Chloride Dihydrate, Potassium Chloride, Magnesium Chloride Hexahydrate, Sodium Lactate Solution, Sodium Hydroxide, Hydrochloric Acid and Water for Injections.

## When should it be used?

When your eyes become red and sore, begin to itch, feel gritty or you feel burning then you have probably developed a dry eye sensation, which needs treatment with Xailin Fresh. Carmellose, the lubricating ingredient alleviates the symptoms of dry eye sensation with its soothing drop.

## How to use:

1. Detach the vial to be used.
2. To open, twist and pull the tab to remove.
3. Tilt your head backwards and look upwards.
4. Gently pull down the lower lid of the eye, insert one drop of Xailin Fresh into the gap between your eye and eyelid and blink a few times to spread the drops evenly over the surface of the eye.
5. Repeat the process for the other eye if you have been told to use the Xailin Fresh in both eyes.
6. Discard the vial after single use.

## Mode of Use:

Detach a single vial

Twist and pull the tab

Gently squeeze the vial to instill the drops

Discard after use

## How much to use?

The usual dosage is 1 drop to be instilled 2 to 4 times a day as needed or as directed by your healthcare practitioner. You can also apply 1 to 2 drops directly on to the contact lens before application. Your healthcare practitioner may advise you to use this product in a different way, so follow their directions about when and how to use this product and read

## EN Instruction for use

It is important to read this leaflet carefully before using Xailin Fresh. The information provided summarises the important points about the product but if you have any further questions then please ask your doctor, pharmacist or optician.




the label. No dosage adjustment is required in children.

## Precautions and warnings:

- This product is for external ophthalmic use only.
- Do not use if the vial is broken.
- Do not use if you are allergic to any of the ingredients.
- If you are unsure about how to use Xailin Fresh talk to your doctor, pharmacist or optician.
- If your condition persists please consult your doctor, pharmacist or optician.
- Do not touch the tip of the vial with your eye or any surface.
- Do not share Xailin Fresh with any other person.
- Xailin Fresh can cause blurring of vision after application. Wait until your vision is clear before driving or using machines.
- Do not use after the expiry date shown on the carton or printed on the label.
- Leave at least 30 minutes between administrations of any other ophthalmic drugs.
- Keep out of the reach and sight of children.
- Do not swallow.
- Do not store above 25°C.
- Do not freeze the product.
- Discard the single dose unit after first opening.
- Consult your doctor, pharmacist or optician if you are pregnant, breast-feeding or have recently undergone an ophthalmic procedure.
- Dispose of the used vial according to your local waste management regulations.

Distributed by **VISUfarma International**  
2405 Route Des Dolines, Drakkar D,  
06560 Valbonne, France  
visufarma@visufarma.com

**Medicom Healthcare Ltd.,**  
235 Hunts Pond Road, Fareham,  
Hampshire, PO14 4PJ, U.K.

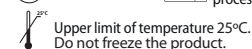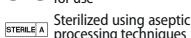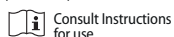Xailin is a trademark of VISUfarma  
Last revision of the text: **February 2017**

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